

Always use gentle pressure when activating the trigger and spraying the cryogen onto the lesion, this will reduce cryogen waste and prevent splatter.

Apply from a distance of approximately 2 to 3 cm.

The table below provides recommended freezing times and recommended number of applications for the commonly treated lesions.

Allow the lesion to thaw completely before freezing again, early thawing will reduce the effectiveness of the freeze and lesion destruction. The recommended treatment times are per lesion per treatment session.

**The healthcare professionals should always assess the individual patient and lesion to determine the correct treatment.**

**Important :** Hydrozid® should only be used by educated healthcare professionals.\*

The total duration of application to a lesion should not exceed 36 seconds.

Lesion	Recommended freezing times It is important to await a complete thaw before freezing again.	Recommend number of applications
Verruca plantaris (plantar warts)	⌚ 6 seconds	4 to 6 times
Verruca vulgaris (common warts)	⌚ 6 seconds	4 to 6 times
Verruca plana (flat warts)	⌚ 3 seconds	6 times
Seborrheic keratosis	⌚ 6 seconds	2 to 3 times
Lentigo (age spots) (facial and non-facial)	⌚ 3 seconds	2 times
Acrochordon (skin tags)	⌚ 3 to 6 seconds	2 times
Actinic keratosis (facial and non-facial)	⌚ 4 to 5 seconds	2 times
Molluscum contagiosum	⌚ 3 to 6 seconds	2 times

Last updated in: 2023/06/01

\*Always read the instruction for use prior to use.



**HYDR**  **ZID**®